


POWDER-LIQUID MIXING SYSTEM









Operating Instructions (using plunger)

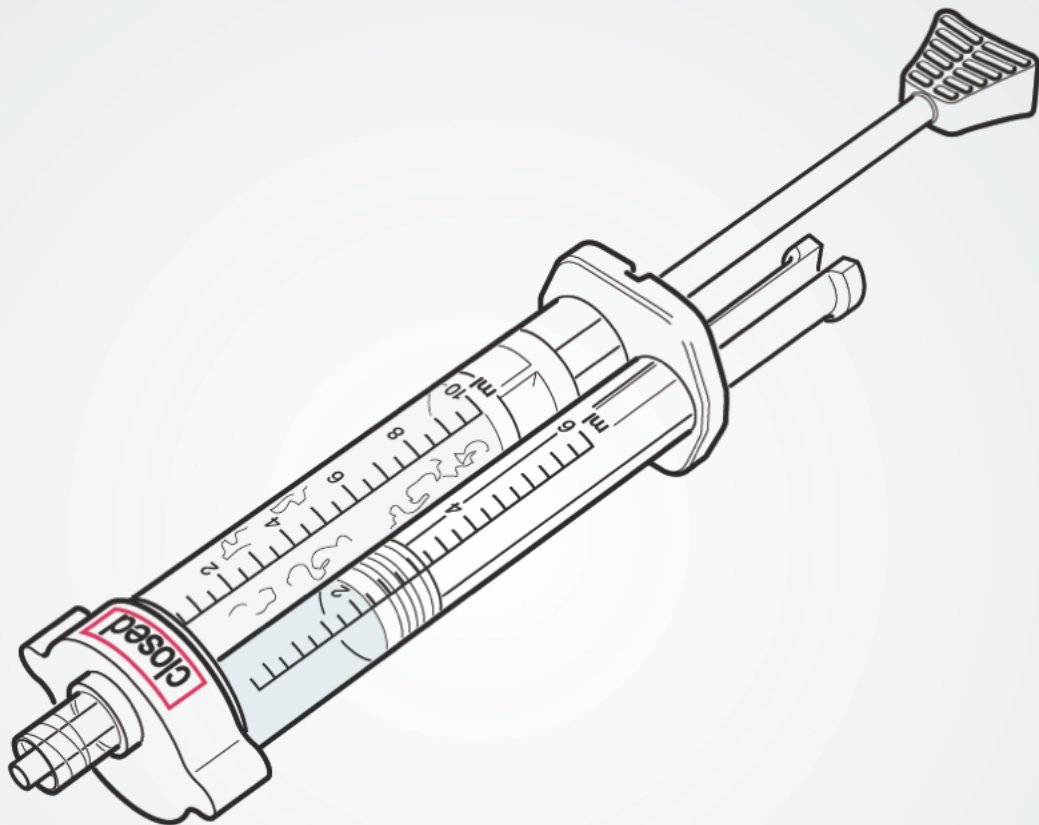

The right mix

MEDICAL MIXING SYSTEMS

Operating Instructions

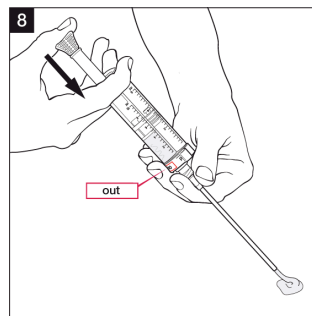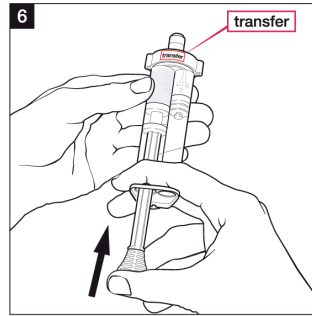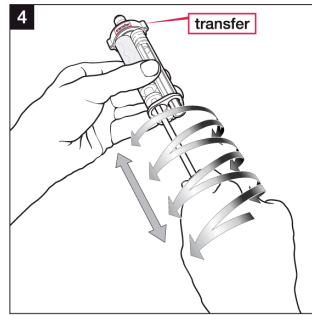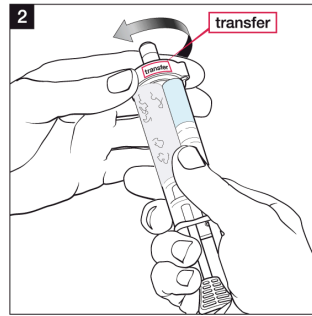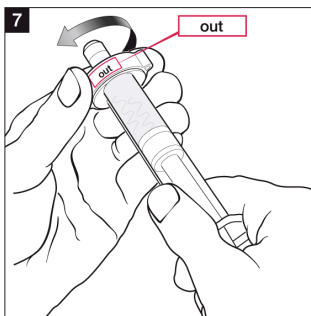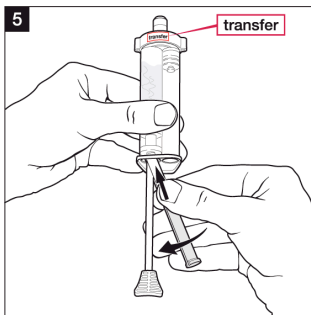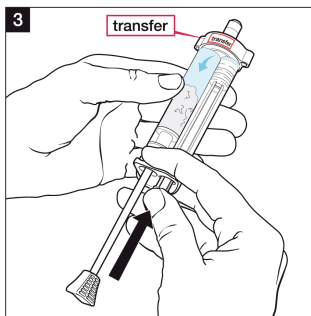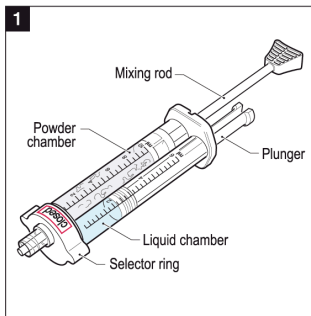

Step 1:

Identification of the system parts.

Step 2:

Turn selector ring from position “closed” to “transfer”.

Step 3:

Push plunger to transfer liquid into powder chamber.

Step 4:

To mix liquid and powder push and pull up to the ends and twist at the same time until completely mixed. Make sure to always twist vigorously when reaching both ends.

Step 5:

Pull mixing rod to the full back position. Remove plunger from liquid chamber and clip plunger onto the mixing rod.

Step 6:

Hold syringe in vertical position with outlet straight up and tap upper end of syringe to bring the air to the top. Slowly push plunger to vent air from powder chamber into the liquid chamber until all air is removed from powder chamber.

Step 7:

Turn selector ring from position “transfer” to “out”.

Step 8:

Attach accessory and push plunger to dispense mixture.